

HydraFacial Cleaning Protocols – Salon-Based Business

1. Pre-Treatment Cleaning & Preparation

- Wash hands thoroughly with antibacterial soap before setting up.
- Disinfect the treatment area, including the treatment bed, chair, and workstation, using a hospital-grade disinfectant.
- Ensure all necessary HydraFacial equipment, including tips, serums, and accessories, are sanitised and prepared.
- Place disposable gloves, masks, and other PPE in an accessible location.
- Lay down disposable bed covers or freshly laundered linens.
- Ensure the HydraFacial machine is functioning properly, with clean tubing and no blockages.

2. Cleaning Protocol During Treatment

- Always wear disposable gloves during treatment.
- Use a fresh, sanitised tip for each client.
- Dispose of used HydraFacial tips immediately after use if using disposable tips.
- Use disposable fibrella for facial cleansing.

3. Post-Treatment Cleaning & Disinfection

- Remove and properly dispose of used gloves and PPE.
- Wipe down the HydraFacial machine, including the tubing, with a medical-grade disinfectant.
- Dispose of the waste jar contents according to local biohazard disposal regulations.
- Sanitise all reusable tools and tips by washing, disinfecting, and sterilising.
- Change and launder all used linens and towels.
- Mop the treatment room floor with a disinfectant solution.
- Restock disposables such as gloves, tips, and bed covers.

4. Waste & Rubbish Disposal

- Used disposable HydraFacial tips: Dispose of in designated medical waste containers.
- Gloves, wipes, and PPE: Place in a sealed waste bin lined with a biohazard bag.
- Liquid waste from the HydraFacial machine: Dispose of as per local waste management guidelines.
- General waste (e.g., packaging, non-contaminated disposables): Dispose of in a standard bin, emptied daily.

Salon-Based Checklist

BEFORE TREATMENT:

- ✓ Wash hands thoroughly
- ✓ Disinfect all surfaces and treatment areas
- ✓ Set up clean, sanitised HydraFacial equipment
- ✓ Prepare disposable gloves, masks, and PPE
- ✓ Lay down fresh linens or disposable bed covers
- ✓ Check the HydraFacial machine for cleanliness and functionality

DURING TREATMENT:

- ✓ Wear disposable gloves during treatment
- ✓ Use a fresh, sanitised HydraFacial tip for each client
- ✓ Dispose of used tips immediately
- ✓ Wipe down the handpiece and machine between applications if needed.
- ✓ Use only disposable facial cleansing tools

AFTER TREATMENT:

- ✓ Dispose of gloves and PPE properly
- ✓ Disinfect HydraFacial machine and tubing
- ✓ Empty and clean the waste jar
- ✓ Sanitise all reusable tools
- ✓ Replace linens and towels
- ✓ Mop the floor with disinfectant
- ✓ Restock necessary supplies